SISSEL® Portable Massage Table Robust

Koffermassagebank
Portable Massage Table
Table de massage portable
Mesa de masaje portable
Lettno da massaggi portatile

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The SISSEL® Robust portable massage table is a practical, functional and sturdy treatment table designed for mobile patient use.

Please read the instructions for use carefully and retain them for the entire lifetime of the product and observe the safety and warning notice.

The portable SISSEL® massage table aids therapists to treat patients at home, at trade fairs, companies, events, at the patient's home and wherever a portable massage table is useful.

Note: We recommend using a purpose-built fixed table for long-term stationary use, for the treatment of obese patients and for special therapies and applications.

Areas of Application:

- Massage
- Physiotherapy
- Manual therapy
- Manual lymph drainage
- Wellness or beauty treatments

Overview of the Main Features:

- Foldable with flexible solid wood struts and coated cables
- Wood parts made from solid beech and laminated beech
- An opening in the front strut makes patient access easier, the back strut adds extra stability
- Working weight approx. 140 kg, static weight approx. 250 kg
- Load-bearing capacity tested according to EN 1725:1998 and EN 1730:2012.
- Washable skin-friendly PU surface, PVC free
- Upholstery: multi-layered PU foam, a viscose foam layer for comfort, rounded and reinforced corners, height approx. 5cm
- Changeable, adjustable head and arm rests
- Carry case
- Measurements: approx. 186 x 71 cm, with head rest approx. 212 cm
- Height: adjustable from approx. 58 cm to 87 cm

Hygiene & Cleaning:

Before use cover the surface with a towel, a loose soft cover or a fitted cover. Wipe the table with neutral soap or a mild disinfectant that contains no more than 30% alcohol.

Storage:

When not in use store the massage table in the case included in a dry place.

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1. Warning notice

- ▮ Check the SISSEL® Robust massage table before each use.
 - All screws and knobs must be securely fastened.
 - Ensure that there are no cracks in the wooden parts.
 - The cables must all be intact.
- ▮ The maximum load should not exceed 250 kg.
- ▮ The feet of the massage table must be adjusted to the same height.
- ▮ Ensure the table has a secure position.
- ▮ Do not insert fingers into the moving parts or joints.
- ▮ Young patients especially babies and toddlers should not be left unsupervised on the table. This product is not suitable for non-responsive patients (risk of falling).

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SISSEL® GmbH